

TOOLS NEEDED

- 2 each – 1/2" open end wrenches
- 1 each – 1/8" allen key
- 1 each – drill with 3/8" bit

NOTE: For reference purposes support mounts come fully assembled. Seat Bases are not pre-drilled to avoid interference with other parts on frame.




STEP 1. Prior to removing OEM seat sling, mark left and right chair frame at the front edge of the OEM seat sling.



STEP 2. Place the front support mount on the frame approx. 2.5" inside the front edge of the original seat base location that was marked.

STEP 3. Place the rear support mount on the frame in the approximate weight bearing location of the user.

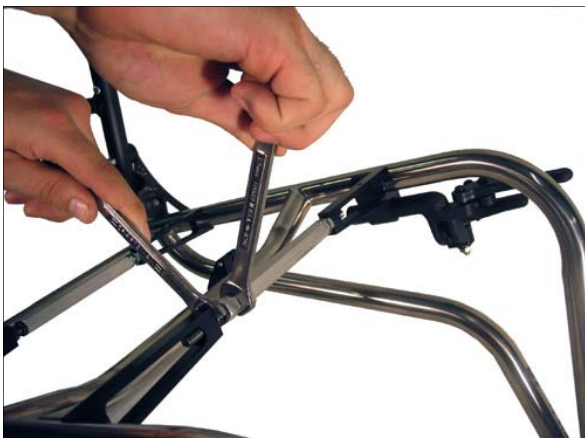

STEP 4. Rotate center hex bar so that the support mounts are "snug" within the frame tubes. Do not over tighten causing flex to the frame.

STEP 5. Place a board or straight edge across the support mounts and take out any buckling.

STEP 6. Tighten jam nuts with the 1/2" wrench.



- STEP 7. Place carbon fiber seat base panel on chair frame and center the edges.
- STEP 8. In order to mount the seat base, you must first mark each hole location through the 10/32" tapped holes in support mount. (use 90° bent scribe or pick to mark holes)



- STEP 9. Drill four 3/8" holes where the seat base is marked and insert the aluminum spacer.



- STEP 10. Tighten the four 10/32" button screw and washers through the holes and spacers into the mounts.

Accessible Designs, Inc.
401 Isom Road, Suite 520
San Antonio, Texas 78216

210.341.0008
210.341.0009 fax
888.684.2234 toll free

email: cs@adirides.com
web: adirides.com

For customer service, call 888.684.2234

For additional information and product images, visit our website - www.adirides.com

About the Carbon Fiber Seating:

Ergonomically designed, the CF Seating promotes an upright posture, easing the strain on the lower back, by providing support at the posterior pelvis. The carbon fiber material is light weight.



- Protect the CF Back from blunt force shocks that may cause damage and/or breakage.
- **Always** check all hardware to make sure they are securely fastened, before using the CF Seat Bases.
- The CF Base may not fit every chair; please consult your clinician or supplier providing your equipment.
- Only use the hardware supplied with the CF Base; do not substitute any other screws, brackets, or clamps.
- 200 lbs. (113.40 kg) user weight limit.

Return/ Repair Policy

All returns require prior authorization from ADI and are subject to a restocking charge. Call TOLL-FREE 1-888-684-2234. Visit our website at www.adirides.com

After authorization has been given, send your CF Back and authorization number to:
ADI, 401 Isom Road, Suite 520 - San Antonio, Texas 78216

Contact Information

If you have any questions concerning repairs or the proper care and use of your ADI product: Call TOLL-FREE 1-888-684-2234 Visit our website at www.adirides.com

Limited 12 - Month Warranty

What is covered?

Accessible Designs, Inc. covers any defects in materials or workmanship of the product. Any substitutions of supplied hardware will void the warranty.

For how long?

12 months from the date the product was originally purchased as a new seatbase.

What we will do.

Within a reasonable amount of time after you return the product to us, we will repair any defect in material and workmanship and ship it back to you, all free of charge.

What we will not do.

We will not repair free of charge a product if it has been misused, damaged by an accident, or damaged from "an act of Nature", e.g., flood, tornado, earthquake, fire.

What the customer must do.

The customer must send the product to ADI with the original sales ticket or other evidence of date of purchase, and pay all delivery costs. It must be sent to ADI, 401 Isom Road, Suite 520 - San Antonio, TX 78216. An original sales ticket is not required if the warranty registration card attached to this manual is completed, returned and is on file at ADI. Failure to submit the original sales ticket or failure to have the warranty registration in our files voids this warranty.

Questions.

Call our Customer Support Department toll free 888-684-2234

Disclaimer

Any implied warranties, including the MERCHANTABILITY and FITNESS FOR A PARTICULAR PURCHASE are also limited to twenty-four (24) months from the date of original purchase and any action for a breach of such warranties or any express warranties herein must be commenced within this time. Accessible Designs, Inc. shall not be responsible for any injuries to property, whether as a result of a breach of express or implied warranties, or otherwise. Accessible Designs, Inc. will not be responsible for the results of careless handling, unreasonable or improper use of this product.

Your State Law Rights and the Effect of this Warranty upon Them

This warranty restricts the duration of all implied warranties to twelve (12) months and limits or excludes recovery for incidental or consequential damages. Some states do not allow limitations on how long implied warranties will last and some states do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty gives you specific legal rights, you may also have other rights which vary from state to state.

Accessible Designs, Inc.
401 Isom Road, Suite 520
San Antonio, Texas 78216

210.341.0008
210.341.0009 fax
888.684.2234 toll free

email: cs@adirides.com
web: adirides.com